

IMPLANT TORQUE GUIDE

Manufacturer	Implant	Torque (Ncm)	Torque Settings	Driver & Head
Astra				
	Astra 3.0	15 Ncm	0.050" (1.27 mm) Hex	«
	Astra 3.5-4.0	20 Ncm	0.050" (1.27 mm) Hex	#
	Astra 4.5-5.0	25 Ncm	0.050" (1.27 mm) Hex	#
	Astra EV 3.0,3.6,4.2,4.8,5.4	25 Ncm	0.050" (1.27 mm) Hex	#
BioHorizons				
	Internal 3.5,4.5,5.7	30 Ncm	0.050" (1.27 mm) Hex	#
	Tapered Internal 3.5,4.5,5.7	30 Ncm	0.050" (1.27 mm) Hex	#
	External Implant 3.5	30 Ncm	0.050" (1.27 mm) Hex	«
	External Implant 4.0,5.0	30 Ncm	0.050" (1.27 mm) Hex	#
Biomet 3i				
	Certain™ 3.25,4.0,4/3,5.0,5/4,6.0	20 Ncm	0.048" (1.22 mm) Hex	«
	Certain™ Prevail™ 3/4/3 (3.4), 4/5/4,5/6/5. 4/3.5/4.6/5	20 Ncm	0.048" (1.22 mm) Hex	#
	Certain™ XP 4/5,5/6	20 Ncm	0.048" (1.22 mm) Hex	#
	External Hex	35 Ncm	0.048" (1.22mm) Hex	#
BlueSkyBio				
	One Stage	30 Ncm	Star & 0.048" Hex	#
	Trilobe	30 Ncm	Unigrip & 0.048" Hex	* #
	Internal Hex & Molar	30 Ncm	0.050" (1.27mm) Hex	#
	Conus 12 & Three	30 Ncm	0.050" (1.27mm) Hex	#
	Max	30 Ncm	0.048" (1.22mm) Hex	#
	Quatro	30 Ncm	0.048" (1.22mm) Hex	#
	OsseoSpeed™ 4.0 - Blue (Previous)	25 Ncm	0.050" Hex	#
Dentsply				
	ANKYLOS® C/X - C/3.5,4.5,5.5, 7.0 (non-indexed)	15 Ncm	1.0 mm Hex	«
	ANKYLOS® C/X - /X 3.5,4.5, 5.5, 7.0 (indexed)	15 Ncm	1.0 mm Hex	#

Manufacturer	Implant	Torque (Ncm)	Torque Settings	Driver & Head
	Friadent	24 Ncm	0.048" (1.22mm) Hex	#
	OsseoSpeed™ EV 3.0 - Green	25 Ncm	0.050" Hex	#
	OsseoSpeed™ EV 3.6 - Purple	25 Ncm	0.050" Hex	«
	OsseoSpeed™ EV 4.2 - Yellow	25 Ncm	0.050" Hex	«
	OsseoSpeed™ EV 4.8 - Blue	25 Ncm	0.050" Hex	«
	OsseoSpeed™ EV 5.4 - Brown	25 Ncm	0.050" Hex	#
	OsseoSpeed™ Profile EV 4.2 Yellow	25 Ncm	0.050" Hex	#
	OsseoSpeed™ Profile EV 4.8 Blue	25 Ncm	0.050" Hex	«
	OsseoSpeed™/OsseoSpeed™ TX 3.0 - Yellow	15 Ncm	0.050" Hex	«
	OsseoSpeed™/OsseoSpeed™ TX 3.5,4.0 -Aqua	20 Ncm	0.050" Hex	#
	OsseoSpeed™/OsseoSpeed™ TX 4.5,5.0 - Lilac	25 Ncm	0.050" Hex	»
	OsseoSpeed™ 4.0 - Blue (Previous)	25 Ncm	0.050" Hex	#
	OsseoSpeed™ TX Profile 4.5,5.0	25 Ncm	0.050" Hex	#
	XIVE® S 3.0	24 Ncm	1.22 mm Hex	#
	XIVE® S 3.4, 3.8,4.5, 5.5	24 Ncm	1.22 mm Hex	«
	FRIALIT™ 3.4, 3.8, 4.5, 5.5	24 Ncm	1.22 mm Hex	«
Dentium				
	Super Line	30 Ncm	0.050" (1.27mm) Hex	#
	Implantium	30 Ncm	0.050" (1.27mm) Hex	«
Glidewell				
	iNCLUSiVE® Tapered 3.7,4.7, 5.2	35 Ncm	0.050" (1.27mm) Hex	«
Hiossen/Osstem				
	HG mini 3.5	20 Ncm	0.048" (1.22mm) Hex	#
	HG Standard 4.0,4.5,5.0	30 Ncm	0.048" (1.22mm) Hex	#
Keystone/Lifecore				
	Genesis & Prima	30 Ncm	Square	■

Manufacturer	Implant	Torque (Ncm)	Torque Settings	Driver & Head
	Renova®	30 Ncm	0.048" (1.22mm) Hex	«
	Restore®	30 Ncm	0.048" (1.22mm) Hex	#
Mega'Gen				
	EZ Plus	35 Ncm	0.048" (1.22mm) Hex	#
	Rescue	45 Ncm	0.048" (1.22mm) Hex	«
	ExFeel Internal 3.5	25 Ncm	0.048" (1.22mm) Hex	«
	ExFeel Internal 4.1,4.8, 5.5	35 Ncm	0.048" (1.22mm) Hex	«
	ExFeel External 3.3	25 Ncm	0.048" (1.22mm) Hex	#
	ExFeel External 3.75,4.0,4.5, 5.0, 5.5	30 Ncm	0.048" (1.22mm) Hex	#
MIS®				
	Biocom & Seven	30 Ncm	0.050" (1.27mm) Hex	#
NEODENT®				
	CM	15 Ncm	0.035" (0.9mm) Hex	#
	CM	32 Ncm	0.048" (1.22mm) Hex	#
	CM	32 Ncm	0.063" (1.6mm) Hex	#
Nobel Biocare®				
	Angulated Screw Channel	35 Ncm	Nobel Biocare® Omnigrip	•
	NobelActive™ 3.0	15 Ncm	Nobel Biocare® Unigrip	*
	Nobel Active™/Nobel Replace™ Conical Connection NP 3.5	35 Ncm	Nobel Biocare® Unigrip	*
	Nobel Active™/Nobel Replace™ Conical Connection RP 4.3,5.0	35 Ncm	Nobel Biocare® Unigrip	*
	NobelReplace™ NP 3.5	35 Ncm	Nobel Biocare® Unigrip	*
	NobelReplace™ RP 4.3	35 Ncm	Nobel Biocare® Unigrip	*
	NobelReplace™ WP 5.0	35 Ncm	Nobel Biocare® Unigrip	*
	NobelReplace™ 6.0	35 Ncm	Nobel Biocare® Unigrip	*
	Bränemark System™ NP 3.3	35 Ncm	Nobel Biocare® Unigrip	*
	Bränemark System™ RP 3.75,4.0	35 Ncm	Nobel Biocare® Unigrip	*

Manufacturer	Implant	Torque (Ncm)	Torque Settings	Driver & Head
	Branemark System™ WP 5.0,5.5	35 Ncm	Nobel Biocare® Unigrip	*
Southern Implant				
	External	32 Ncm	0.048" (1.22mm) Hex	#
	Tri Nex	32 Ncm	Unigrip	*
	Octa	32 Ncm	Star/Torx	
Straumann®				
	Bone Level 3.3 NC (Narrow CrossFit™)	35 Ncm	SCS Screwdriver	*
	Bone Level 4.1,4.8 RC (Regular CrossFit™)	35 Ncm	SCS Screwdriver	*
	Standard/Standard Plus 4.8 RN (Regular Neck)	35 Ncm	SCS Screwdriver	*
	Standard/Standard Plus 4.8 WN (Wide Neck)	35 Ncm	SCS Screwdriver	*
	Standard Plus 3.3 NN (Narrow Neck)	35 Ncm	SCS Screwdriver	*
	Tissue Level	35 Ncm	Star/Torx	
Sybron				
	Pitt Easy	25 Ncm	0.067" (1.7mm) Hex	#
Thommen				
	SPI® 3.5	15 Ncm	Four Lobe	
	SPI® 4.0,4.5, 5.0, 6.0	25 Ncm	Four Lobe	
	SPI® Zirconia Abutment	20 Ncm	Four Lobe	
Zimmer Dental				
	Tapered Screw-Vent 3.5,4.5, 5.7/Tapered Screw-Vent 3.5,4.5,5.7 friction fit	30 Ncm	(.050) 1.25 mm Hex	#
	Screw-Vent 3.3, 3.7,4.5/Screw-Vent 3.3, 3.7,4.5 friction fit	35 Ncm	1.25 mm Hex	•
	SwissPlus™4.8	35 Ncm	1.25 mm Hex	#



.048" (1.27mm) Hex

For Noix JBlocare, Homet 3L
Fridant, Keystone,
MegaCervHouer



.05" (1.27mm) Hex

OST (1.27mm) Hex Long
For Zhn met Astra, BtoHortzoreJime-Lodt
Mit, SteiCSs, Sybrorv hiplanCfrectPetoSeil
DantjuiTvöldBM^l



PfiOC Driver - PROC Driver Long
StarDiverfw Nobd HooereActhe Blue
Sky Bio & Unfrctp Screw, Neoss



m Driver ■ m Driver Long TTI
Driver for Strau man π 5CS



LOC Triangle Driver
LOC Triartgle Driver Long
Far LocBtar^AbutiTient



.07" (1.7mm) Hex
For Paragon TSI Abutment




.035" (0.9mm) Hex
.05" (1.27mm) Hex Long
FoiNoWBlocere^l.Keystixie^M
Fridant. PerioSeal



aer (Ziumm) Female He* For
NobeBoore, 31



TG Driver
For 31TG Abutment, Strainwm SCS

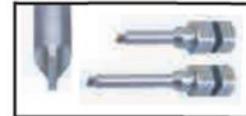

Screw Driver - Screw Driver Long
Flat-Blade Screwdriver



.OF (1.25mm) Square-Ü5"
(1.25mm) Square Long For
Keystone, 31



Vertrieb / Distribution
Deutschland, Belgien,
Niederlande, Luxemburg,
Frankreich:

RUNDAS GmbH
Amalienstraße 62
46537 Dinslaken

Tel.: 02064 625 95 50
Fax: 02064 625 95 80

E-Mail: info@rundas.de
Internet: www.rundas.de